



How to Download your HealthBux iSync Pedometer Software.


1) When you click on the download link, you will see two security warnings. Each time you see a warning, **click "RUN."**

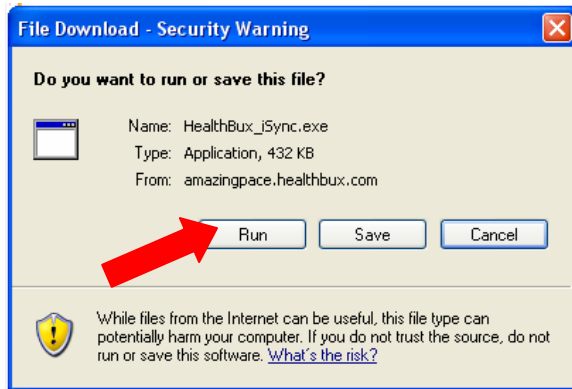


3) Next you will see a box that says HealthBux isync is now installed. **Click OK.**

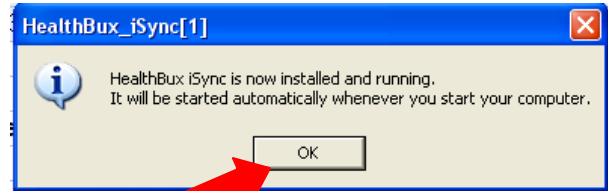

4) Then you will see a final security warning with a green check mark in a box. **Click on the green check mark to remove it so you do not get the security warning again. Make sure the box is empty, then click RUN.**

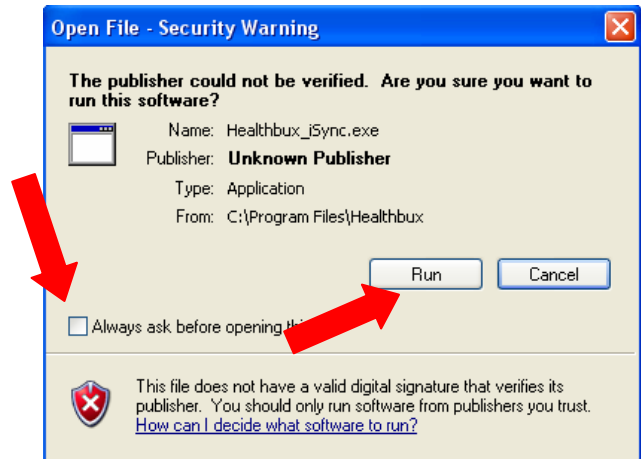

2) Next you will be asked if you want to install the software. **Click YES.**

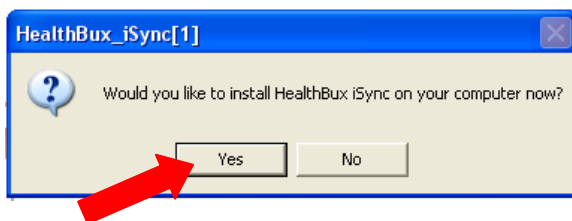

5) Check the bottom right hand corner of your screen, near the time. You should see the HealthBux icon. (You may need to click on a button to expand your tray to see all the icons).



Congratulations! Your HealthBux iSync installation is complete!